

TwinLoop FLEX™

Instrumentation System



TwinLoop FLEX

Instrumentation System

The TwinLoop FLEX Instrumentation System is designed to provide easy access along the glenoid rim during arthroscopic instability repair. Paired with Stryker's TwinLoop anchors, the flexible instrumentation is an excellent solution for surgeons who desire simplification of instability repair.



The tips of the drill guides are designed with a window, for easy arthroscopic viewing of laser lines for depth control and eyelet orientation.

Enhancement

of Instability Repair Procedures

TwinLoop FLEX Drill Guides

The TwinLoop FLEX Instrumentation System contains five different drill guides. The standard and reverse mouth drill guides are designed with a 12° or 25° curved tip and a straight option is also available. These guides are designed to enable the surgeon to reach farther around the glenoid rim to more securely position the anchor. In addition, the need to create an inferior anterior portal is eliminated, therefore helping to reduce the risk of damaging soft tissue adjacent to the inferior glenoid.




TwinLoop FLEX

Instrumentation System

TwinLoop FLEX Obturators

Two different obturator options are available for the surgeon to choose from, based upon preference. The bullet tip and trocar tip obturators are both flexible and can be used universally with any of the five drill guides.

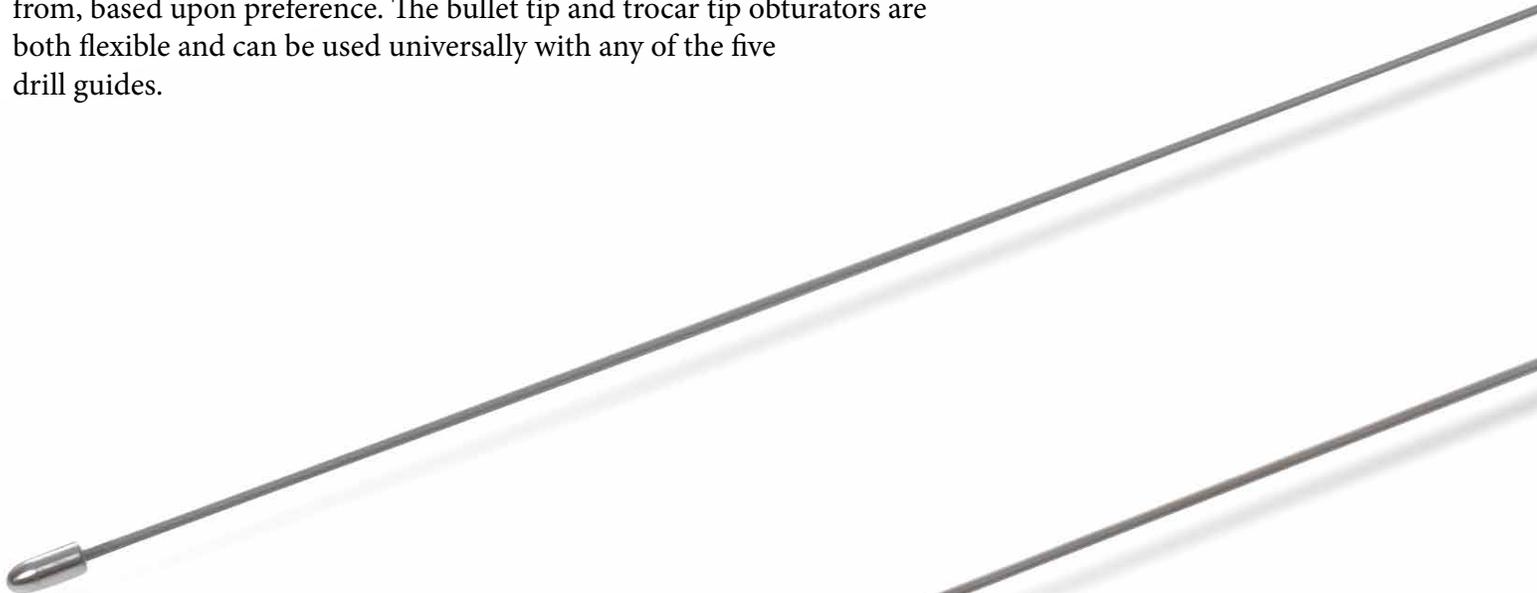

Trocar Tip



Bullet Tip





TwinLoop FLEX Drill

The Flexible Drill is designed to fit smoothly through all five of the drill guides, for more precise pilot hole placement, while maintaining the use of traditional anterior and posterior portals.




TwinLoop FLEX

Instrumentation System

TwinLoop FLEX Anchors

Stryker's PEEK TwinLoop Anchor is now available on an inserter indicated for the TwinLoop FLEX system. The PEEK TwinLoop anchor features one strand of #2 Force Fiber high-strength suture threaded through the body of the anchor.

The body of the PEEK TwinLoop Anchor follows a wedge design of 2.4mm to 2.9mm designed for optimized fixation. With incremental barbs, the overall size of the anchor is 3.5mm, excellent for SLAP and Bankart procedures.

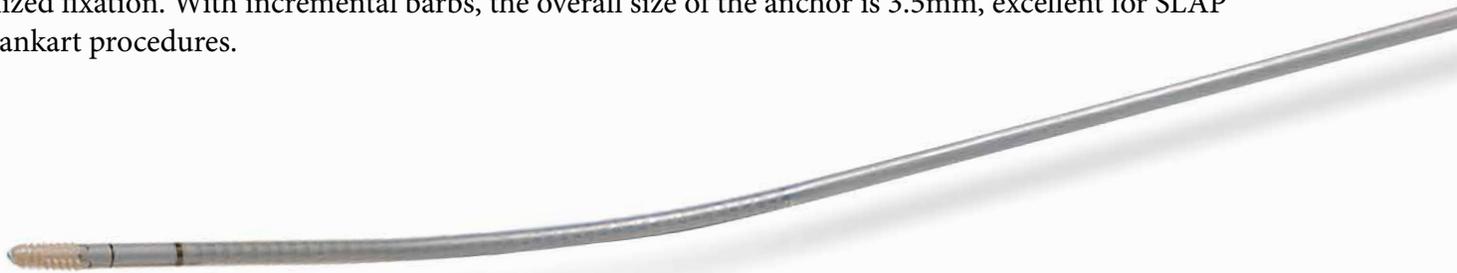

**PEEK TwinLoop Anchor with
One Strand of Force Fiber
3910-405-638**



Description	Catalog Number
TwinLoop FLEX Drill	3910-400-100
TwinLoop FLEX Bullet Tip Obturator	3910-400-101
TwinLoop FLEX Trocar Tip Obturator	3910-400-102
TwinLoop FLEX 12° Guide	3910-400-103
TwinLoop FLEX 12° Reverse Mouth Guide	3910-400-104
TwinLoop FLEX 25° Guide	3910-400-105
TwinLoop FLEX 25° Reverse Mouth Guide	3910-400-106
TwinLoop FLEX 0° Guide	3910-400-107
3.5mm PEEK TwinLoop FLEX One Strand #2 Force Fiber	3910-405-638

Joint Replacements

Trauma, Extremities & Deformities

Craniomaxillofacial

Spine

Biologics

Surgical Products

Neuro & ENT

Interventional Spine

Navigation

Endoscopy

Communications

Imaging

Patient Care & Handling Equipment

EMS Equipment

325 Corporate Drive
Mahwah, NJ 07430
t: 201 831 5000

www.stryker.com

A surgeon must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. Stryker does not dispense medical advice and recommends that surgeons be trained in the use of any particular product before using it in surgery.

The information presented is intended to demonstrate the breadth of Stryker product offerings. A surgeon must always refer to the package insert, product label and/or instructions for use before using any Stryker product. Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker products in your area.

Stryker Corporation or its divisions or other corporate affiliated entities own, use or have applied for the following trademarks or service marks: Stryker, TwinLoop FLEX. All other trademarks are trademarks of their respective owners or holders.

Literature Number: **LJPTLF-B Rev. 1**
MS/GS 06/10

Copyright © 2010 Stryker
Printed in USA