



The IDEAL™ Suture Management Family of arthroscopic devices includes method choice options (retriever or shuttle), tip configuration, and actuation, all with technique versatility in a comprehensive, disposable instrumentation system.



IDEAL™ Suture Shuttle with CHIA®

Strength: Both the shuttle shaft and tip provide the surgeon with assured confidence during arthroscopic procedures.

Ergonomic Design: The patented handle configuration allows for a more controlled grip and tactile feel.



SLAP and Bankart Repairs



SLAP and Bankart Repairs



Bankart Repairs



Bankart Repairs



SLAP Repairs and/or Rotator Cuff Suture Management



Left or Right Suture Management



Penetrating Biceps, Plication Stitches, Side-to-side in Rotator Cuff




Access: Percutaneous approach through 18g spinal needle improves access to targeted anatomy

Procedural advantage: Creates a step-wise approach to suture storage and organization

Percutaneous advantage: Minimally invasive, reduced trauma with improved access

CHIA PERCPASSER® gives you choice

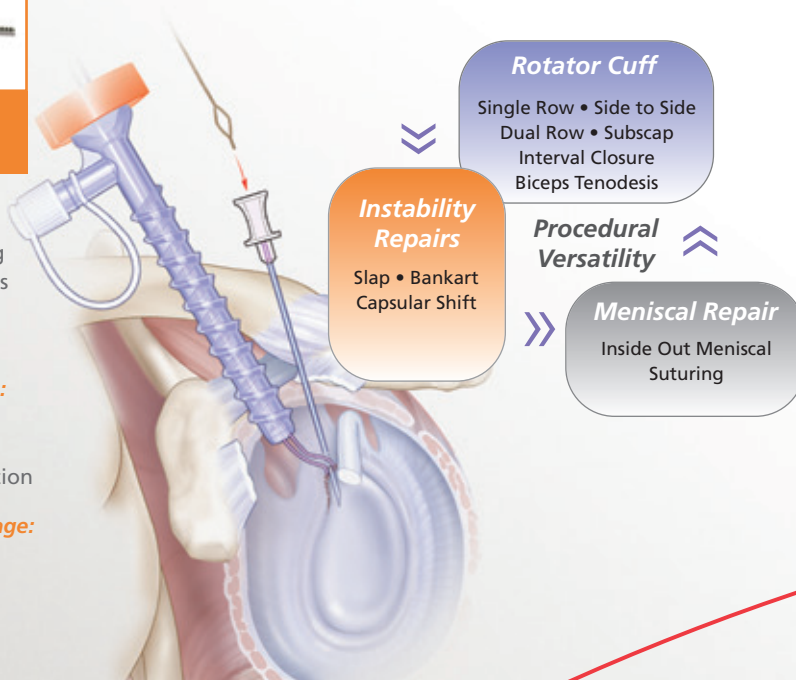




Rotator Cuff
Single Row • Side to Side
Dual Row • Subscap
Interval Closure
Biceps Tenodesis

Instability Repairs
Slap • Bankart
Capsular Shift

Procedural Versatility
Meniscal Repair
Inside Out Meniscal Suturing

IDEAL™ Suture Grasper

Versatility: Enables passing or retrieval of suture through tissue without bite size restrictions.

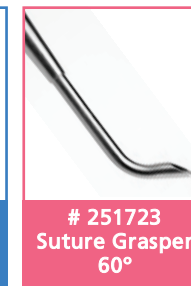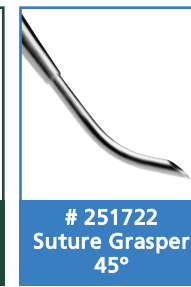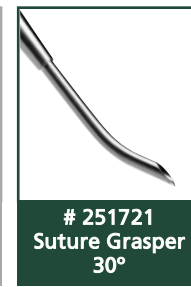


never stop moving™



IDEAL™ SUTURE SHUTTLE with CHIA®

The newest addition to the IDEAL™ Family addresses ease of handling and surgical finesse to meet the demands of interoperative procedures.

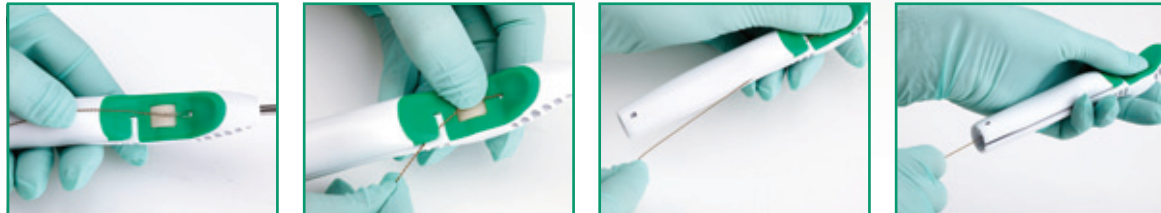


Loading the IDEAL Suture Shuttle: • Insert the tip of the CHIA PERCPASSER® kite into the hole just above the wheel.
• Place your thumb firmly on the CHIA and wheel; with your other hand, firmly snap the length of the CHIA into the side loading gate of the shuttle. • Place your thumb on the wheel (and CHIA) and begin to advance the CHIA up through the end of the needle tip.



IDEAL Suture Shuttle
with CHIA®

Removing the IDEAL Suture Shuttle
Pull back on the tail of the CHIA until the CHIA kite with suture collapses back into the needle tip. Holding the CHIA in place, back the IDEAL Suture Shuttle out of the working portal.

Surgical images provided by Dr. Ed Tillett, Louisville, KY.

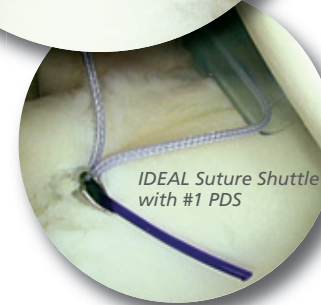

IDEAL Suture Shuttle
with #1 PDS

Versatility and Options in Shuttling
Allows for the use of an alternate shuttle option like #1 PDS.

IDEAL™ SUTURE GRASPER



1. The IDEAL Suture Grasper in the "CLOSED" position – when piercing tissue



2. In the "SLIP" position – allows suture to slide while it is retrieved



3. In the "OPEN" position – exposing the nitinol hook to capture the suture.

The IDEAL Suture Grasper Advantage This single use, sterile, packaged device offers consistent, reproducible tissue penetration with speed and versatility.

never stop moving™